INSTRUCTIONS FOR USE

RapportTM
PREMIER
IMPOTENCE MANAGEMENT SYSTEM

A Vacuirect Product

 **OWEN MUMFORD**

www.owenmumford.com

UK

Owen Mumford Ltd, Brook Hill, Woodstock,
Oxford OX20 1TU, England
T : 01993 812021 F : 01993 813466
E : info@owenmumford.net

USA

Owen Mumford Inc, 1755-A West Oak Commons Court,
Marietta, Georgia 30062, USA
T : 800-421-6936 F : 770-977-2866
E : info@owenmumfordinc.com

FRANCE

Owen Mumford, BP 444, 27204 Vernon Cedex, France
T : 02 32 51 88 70 F : 02 32 21 96 95
E : owenmumford@compuserve.com

GERMANY

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
Owen Mumford GmbH, Sternplatz 3,
D-97896 Freudenberg/M, Germany
T : 09375 92990 F : 09375 929929
E : info@owen-mumford.de

EXPORT

Owen Mumford Ltd, Brook Hill, Woodstock,
Oxford OX20 1TU, United Kingdom
T : +44 1993 812021 F : +44 1993 813473
E : export@owenmumford.co.uk

*Rapport Premier V.T.D. is manufactured in South Africa
by Vacurect Manufacturing (Pty) Ltd*

Patent Application Number: PCT/US99/04736

SM/2200/11/0000/02

Contents/Contient/Inhalt

- 4** Risks
- 5** Warnings
- 6-7** How the Rapport Premier Vacuum Therapy Device works
- 8-9** Choosing the correct ring/Using the pump to create an erection
- 10** Cleaning instructions
Device maintenance
- 11** Important precautions
Some special points to remember
- 12** Guarantee

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Risks

There are some risks in using a vacuum therapy device.

The most common ones experienced are:

Bruising of the penis. This often happens when you first use the device and can range from small red spots on the skin of the penis up to painful black or blue bruises. This is best avoided by increasing the amount of vacuum pressure slowly when first using the Rapport Premier V.T.D.

Pain in the penis. This can occur when the vacuum pressure is applied. If this does happen, stop pumping immediately and release the pressure using the pressure release valve. If the pain comes back consult your doctor before using the device again.

Warnings

Cautionary Points on the Use of Vacuum Therapy Devices.

- Use the minimum amount of vacuum necessary to achieve an erection. This will prevent bruising.
- Stop drawing vacuum if pain occurs.
- Caution is advised when using a vacuum pump following removal of a penile implant because of possible surgical scarring and subsequent impairment.
- Misuse of a vacuum pump may aggravate already existing medical conditions such as Peyronie's disease, priapism and urethral stricture.
- Use of the vacuum pump may bruise or cause rupture of the blood vessels immediately below the surface of the skin, which may produce hemorrhage or haematoma.
- Do not use the vacuum pump while under the influence of alcohol or drugs.
- If any of the above problems occur, discontinue the use of the vacuum pump and consult your doctor.

How the Rapport Premier Vacuum Therapy Device Works

An erection is created, by drawing blood into the penis by means of a vacuum. The erection is maintained by placing a ring at the base of the penis, which ensures that the blood remains there until the ring is removed.

The ring should be a snug, comfortable fit without being too tight.

The use of the Rapport Premier Vacuum Therapy Device is not intended to replace your doctor's diagnosis of the underlying cause of your impotence, but to help achieve and maintain an erection. It is always best to discuss your individual problem with a doctor before using this treatment.

“Studies have shown that by the use of vacuum devices, in excess of 80% of men are able to achieve a degree of penile tumescence sufficient to allow completion of sexual intercourse.”

The Journal of Sexual Health June 1993 Volume 3 Number 6

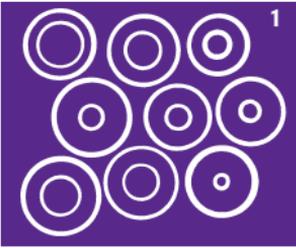
Initially it may take some time to familiarise yourself with the Rapport Premier V.T.D., so please take the time to read this instruction booklet and to practice the technique of creating and maintaining the erection, before you attempt sexual intercourse. If you have not had an erection for some time, you may not be able to create one straight away. It may take several practice sessions to achieve an erection for the first time.

If you have not had a firm usable erection for more than one year, you will need to practice even more. With patience and daily practice for 20-30 days you should find success.

The Rapport Premier comes complete with everything you need for immediate use.

The discreet carrying bag contains:

1. Nine individually packaged and numbered rings.
2. A clear durable penile tube.
3. A tube of water soluble lubricating jelly.
4. Bottle of silicone oil.

A video presentation is available which shows a demonstration of the Rapport Premier being used to create an erection. The video can be purchased, but is refundable on return against the purchase price of the Rapport Premier V.T.D.

Choosing the correct ring

One of the most important parts of the Rapport Premier V.T.D. is the ring. The ring must encircle the penis and fit comfortably tight. The ring works by retaining the blood in the penis to maintain a firm erection.

First of all, you need to find a ring size that suits you best. Using the card ring sizing guide provided, place your flaccid (non-erect) penis into the slots until you find one which fits the circumference of your penis. Each slot has a ring size marked above it which corresponds to the ring you require, the most common ring size is 18. Please do not choose a ring based on its hole size, use the sizing guide.

Attach the ring to the open end of the penile tube. Then by pressing the lip of the ring against the edge of the tube you will be able to form an airtight seal (*fig. 1*).

Using the pump to create an erection

1. Apply a sufficient amount of water soluble lubricating jelly around the inside rim of the ring and to the penis (this will ensure that the penis slides quickly and smoothly through the ring and does not stick to the side of the penile tube) (*fig. 2*). ***Silicone oil must not be used as it will damage the rings.***

2. Next, hold the penile tube with the lubricated entrance of the ring against the tip of the penis and start pumping slowly by sliding the outer cuff of the penile tube back and forth. ***Please take care not to pump too fast as this may bruise or cause a rupture of the blood vessels immediately below the surface of the skin.*** Initially you will have to do up to 15 full strokes. This will smoothly pull the penis through the ring and into the tube. (You need to do full strokes for maximum efficiency and silent operation) (*fig. 3*).

3. Once the penis has been pulled into the tube, the tube may be firmly pressed against the body. Stop pumping and wait for a full erection to develop.

4. Depending on the initial state of the erection, this will typically take between 30 seconds to 5 minutes.

If a full erection has not developed in 5 minutes a few extra pumping strokes may be required.

5. When a satisfactory erection has been developed the pump may be released by gently squeezing down on the ring, thereby instantly releasing the vacuum. The tube may then be removed, leaving the ring at the base of the penis to maintain the erection.

6. With the ring in place a lasting erection should now be maintained and sexual intercourse can be accomplished.

NOTE: If a condom is to be worn, it should be put on as a last step. Do not wear a condom during the vacuum process.

7. If the penis becomes soft within 5 minutes you need to use a smaller size ring. Work your way down the sizes until you find a ring that is comfortably tight but no tighter than necessary.

8. If more rigidity is required after some use in intercourse, further pumping may be resumed without removing the ring.

Caution - the ring must not be left in place for more than 30 minutes.

Before removing the ring it is recommended that the penis be squeezed firmly for about 10 seconds to reduce the blood pressure in the penis in order to ease removal of the ring. To remove the ring, firmly grip with both hands on the outside rim and then gently pull it away from the body and off the penis (*fig. 4*).

Cleaning Instructions

Clean the pump by hand washing with cold water only so as not to remove the silicone oil from the sliding surfaces too quickly. It should normally be sufficient to wash only the open end surfaces of the pump.

Always ensure the constriction ring is thoroughly cleaned.

Device Maintenance

For continued smooth operation of the pump it is essential that all sliding surfaces and the two O-rings be lightly lubricated about once a month. For this purpose make sure that only pure, plastic friendly silicone oil is used. Only a few drops are required per service.

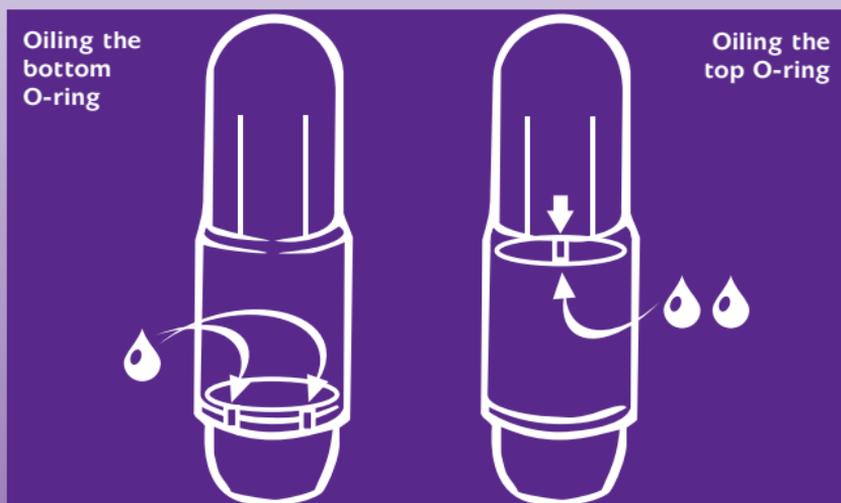
Bottom O-ring - place 1 drop of silicone oil into each of the 4 notches and then rotate the cuff at least 2 complete turns to distribute the silicone oil evenly around the ring.

Top O-ring - look for the arrow marked midway down the device. At this point add 2 drops of silicone oil down inside the cuff so that it drips onto the O-ring. Rotate the cuff at least 2 complete turns to distribute the silicone oil evenly around the ring.

Additional silicone oil may be added to the outer surfaces to prevent scratching but only a few drops are required.

Do not use the wrong oil as this may result in stickiness and/or build up of dirt and eventual malfunction of the product. Replacement oil can be purchased separately from Owen Mumford.

Never use the enclosed silicone oil or any petroleum based oil on the rings as this will cause permanent damage to the rings.

Do not damage or scratch any of the sealing surfaces since this will drastically reduce the pump efficiency.

Take care not to damage the screw threads.

Store in a dark, dry place.

Important Precautions

Never wear the ring for more than 30 minutes.

Never over pump. The user should always apply only the minimum amount of vacuum necessary to achieve a satisfactory erection. If you experience pain when you use the pump release the vacuum by removing the tube from the ring. The device has been designed to limit the vacuum to about 0.57 bar, which is considered a safe maximum value. The typical operating vacuum is approximately 0.4 bar which should be reached within about 15 to 20 full strokes.

Do not wear the ring whilst under the influence of drugs or alcohol.

Do not risk falling asleep whilst wearing the ring.

Owen Mumford will not be liable for any misuse of this product because a user failed to read and comprehend the instructions.

Some Special Points to Remember

We recommend that the first few times you use the Rapport Premier V.T.D. that you do not make the penis any harder than necessary. The penis may not be used to that amount of pressure, size and hardness. Increase the firmness of your erection gradually.

With the ring in place the penis may have a bluish colour and the head may appear larger than normal. Do not be alarmed, this is normal when the penis is filled with blood which is contained by the ring.

Remember whilst wearing the ring ejaculation does not necessarily occur as heavily as normal.

You can obtain a second erection, but you must make sure you wait at least 60 minutes between uses of the ring.

Warranty

The Rapport Premier V.T.D. is covered by a guarantee (excluding rings, water soluble lubricating jelly and oil).

Please ensure you fill in the date of purchase on the guarantee below.

In the unlikely event of a part failure, return the part to Owen Mumford quoting the Lot Number and date of purchase from the guarantee.

G U A R A N T E E

Rapport Impotence Management System

CODE No: SM2200

The Rapport Premier V.T.D. is covered by a guarantee (excluding the rings, water soluble lubricating jelly and oil).

Please keep this safely. If you need to contact us for any reason you must quote the Lot Number and date of purchase to validate this guarantee.

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