## Peak Flow Meter

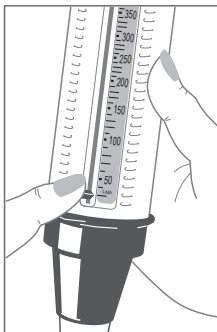
**EN**  
YOUR PEAK FLOW METER  
Your Mini-Wright Low Range Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your Peak Flow Meter, and how to record the readings. Peak flow measurements are an indication of how well air is moving through the airways in your lungs - changes in your peak flow may require special treatment, because they indicate what is happening with your lungs.

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**1**  
Insert the mouthpiece into the meter. Ensure the pointer is at the bottom of the scale.

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**2**  
Hold the Peak Flow Meter so that your fingers are clear of the scale and slot. Do not obstruct the holes at the end of the Peak Flow Meter.

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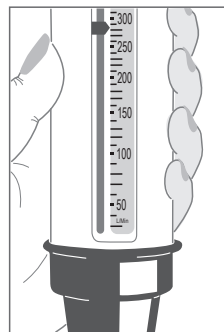
**3**  
Stand up if possible. Take a deep breath, place the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can. Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.

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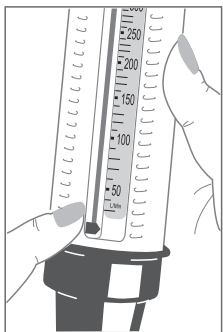
**4**  
Note the number on the scale indicated by the pointer.

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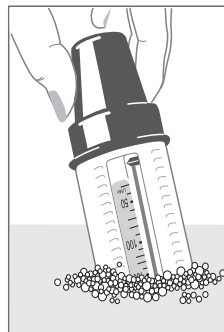
**5**  
Return the pointer to the bottom of the scale and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your Peak Flow Diary.

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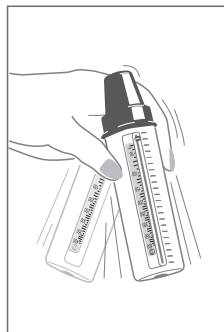
**CLEANING YOUR PEAK FLOW METER**  
- Single Patient Use  
Immerse Peak Flow Meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning. Rinse in clean warm water and shake gently to remove any excess water. Allow to dry thoroughly before using again.

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**CARE AND USE OF YOUR PEAK FLOW METER**  
This peak flow meter does not require maintenance (other than cleaning) or calibration for the three years following its first use. At the end of three years, it should be replaced. When not in use, this device should be kept in a clean, dry area away from any hazardous substances.

It is important to keep your Peak Flow Meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time. The mouthpiece is detachable and should be kept clean using a proprietary disinfectant solution, such as Dettol. Fungal growths may occur if the instrument has not been cleaned for some time.

The Mini-Wright Low Range Peak Flow Meter is suitable for single patient use. The Mini-Wright Low Range Peak Flow Meter is suitable for multi patient use if Clement Clarke's validated cleaning instructions are followed. These are available on request, or via the internet at [www.clement-clarke.com](http://www.clement-clarke.com).

**PERFORMANCE ACCURACY**  
The Mini-Wright Low Range Peak Flow Meter (Standard Range) meets NAEP guidelines and is designed to meet the accuracy required within ISO 23747 for the range 30 to 400 L/min.

Mini-Wright peak flow meters that have been calibrated to the ISO 23747 standard, feature a yellow scale, and have an accuracy better than  $\pm 10$  L/min or 10% of the reading, whichever is the greater. All readings are expressed as BTPS values. The maximum resistance to airflow is 0.35 kPa/L/min at 100 L/min.

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