**LFM**  
**Bruksanvisning**  
**Instructions for use**  
**Brugervejledning**  
**Gebrauchsanweisung**  
**Instrucciones de uso**

Fig 1.

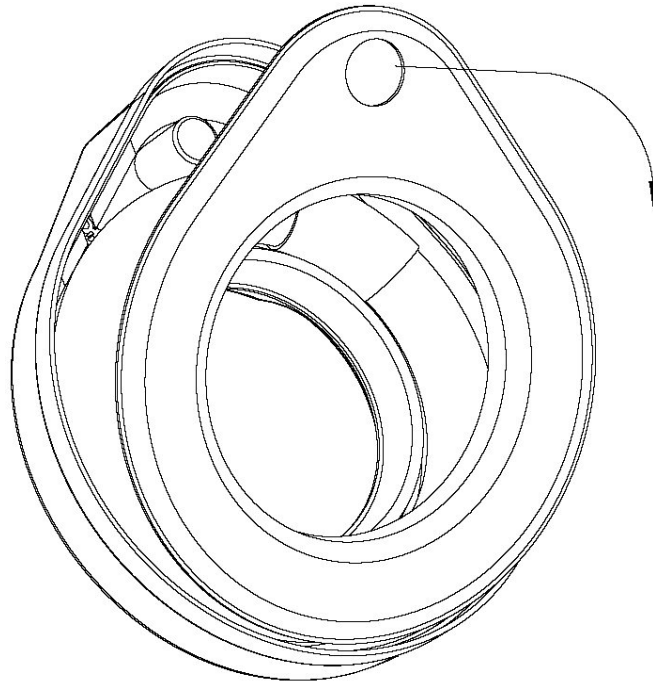
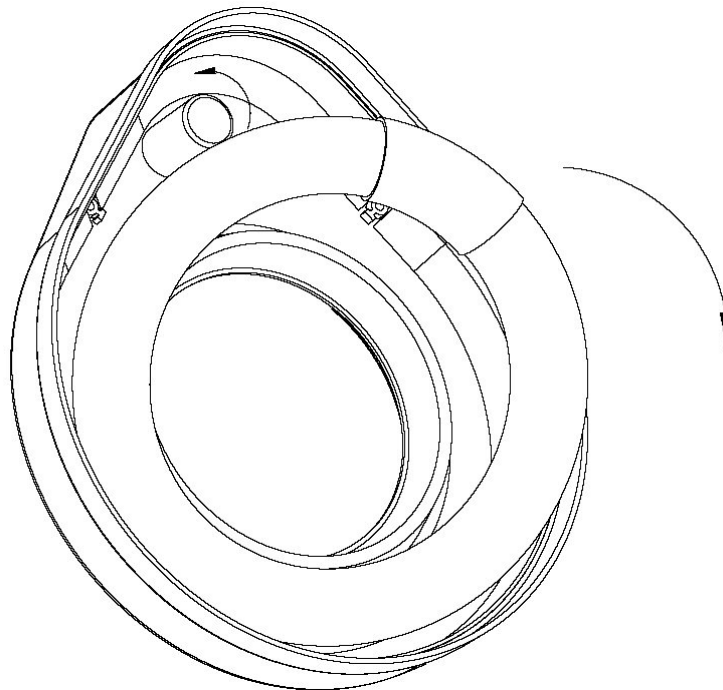
Fig 2.

**Fig 3.**

Assemble/ disassemble of suncover:

**Fig 4.**

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## ENGLISH

Instructions for use - LFM and LFM Medical  
Illuminated magnifier - 22 W

### General information

Magnifying lamp with circular T8 light tube.

### Installation:

This product contains a spring loaded arm. In order to avoid damage or possible injury, first mount table bracket or floor stand. Insert product and remove protective plastic sleeve or rubber band, and carefully unfold the product.

For dismounting/mounting of sun cover and addition lens please see page 3.

Always use enclosed table bracket, article number 8990-010 or optional floor stand 8990-028.

Luxo cannot guarantee the lifetime and functionality if third part accessories are used.

### Precautions:

- Avoid spilling liquid on the lamp head
- NB! Never open the lamp head while the current is connected.

### Replacing the light source

Start by switching off the power at the mains and removing the plug.

1. Remove the light source cover (see fig 1)
  2. Use both hands to remove the light source (see fig. 2)
  3. Remove the starter by unscrewing it counter clockwise (see fig. 2)
- Fit the new light source by reversing the above procedure.

### Cleaning

Open the lamp head as described above. Blow and dry off any dust and dirt.

Using of different cleaning agents for cleaning this product:

Avoid use of alcohol, acids or any kind of organic solvents (Isopropanol etc.)

Only use the weak mix of ordinary soap and water without any perfumes.

### Troubleshooting

1. The luminaire does not work.
  - Replace the circular 22W T8 light tube
- 2 For qualified electrical fitters and other similarly qualified personnel
  - Check the mains connection
  - Replace the fuses (NB! Only on MEDICAL version)
    - ⇒ Remove the lamp from mains supply.
    - ⇒ Loosen the screw and remove ballast box cover
    - ⇒ Exchange fuses
    - ⇒ Replace ballast box cover, and tighten the screw

Should there arise other problems please contact your local LUXO representative.

### Classification

	LFM	LFM Medical
Medical class:		Class 1
Degree of protection against electric shock	Class I (grounded)	Equipment type: B Applied Part
Permitted current leakage		Max 0.5 mA
Protection reliability		Resistance in earth circuit - earth connection 0.1
Power supply	230AC 50 Hz	230AC 50 Hz
Power consumption	Max. 95VA	Max. 95VA
Mode of operation		Continuous
Protection against risk of explosion	Not protected	Not protected
Protection against liquid penetration	IP20	Not protected
Degree of mobility	Mobile table	Mobile table
Type of approval	ENEC 17	MEDICAL Standard
Safety test in line with	EN 60598-2-4(1997), EN 60598 1(2004)+A11(2000)+ A12(2002)	IEC/EN 60601-1(1990) 2 Ed +A1(1993)+A2(1995)+ A13(1996)+CORRIGENDIUM(1 994)
EMC test in line with	EN 55015(2000)+A1(2001)+A 2(2002)	IEC/EN 60601-1-2(2001)
Primary fuse	No fuse	IEC 127-5X20mm, 2xT0.4AH 250V

### Safety

Users of this form of lighting should be aware that the manufacturer of the product can only accept liability for the lighting when maintenance, repairs or changes are carried out by the manufacturer or by companies which the manufacturer has approved to carry out such work, and when components that influence the safety of the lighting are replaced with original spare parts. The lighting is not intended for use in areas where there is a risk of explosion.

This product complies to IEC/EN 60601-1-2(2001), electromagnetic compability for medical devices. Electromagnetic or other interference between this equipment and other devices should not occur. In the event of interference, please move the equipments apart from each other.

The power cable must only be replaced by a qualified electrical fitter or other similarly qualified professional.

Any repairs of the arm must be performed by LUXO or LUXO approved personnel. The arm contains high tensions springs.

### Maintenance

Cleaning of product and replacement of light source is the only maintenance needed.

**Environmentally conditions:**

Transport and storage

Ambient temperature ..... -20° C to 70° C  
 Moisture content ..... 10 % to 100 % (keep dry)  
 Atmospheric pressure ..... 500hPa til 1060hPa

Operate

Ambient temperature ..... 5° C to 40° C  
 Moisture content ..... 30 % to 75 %  
 Atmospheric pressure ..... 700hPa to 1060hPa

**Symbol explanations:**

: Do not throw away. Electronic waste. The product must be recycled

: Self declaration symbol

: Approved by Nemko

: Approved by Nemko

: Indicates equipment type: B Applied Part


: O = light off, I = light on

: Warning, read user manual before use.

: Keep away from sunlight. Lens could potentially become a burning glass.

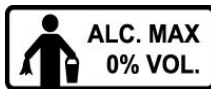: Do not use alcohol based cleaning solutions

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